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GB IMPORTANT INSTRUCTIONS – PLEASE READ CAREFULLY BEFORE USE.

The Special Teat Range from NUK – For children with a cleft lip or cleft palate.

NUK has developed a special teat range, in the original NUK orthodontic shape, for children with a cleft lip or cleft palate. These teats are designed as a feeding aid and can be used with any standard neck bottle. It is precisely when feeding and drinking aids are required that physiological and psychological needs are often not met.

For newborn babies affected by these conditions ingestion can be associated with considerable problems. The reason for this is the inability of these infants to generate an intraoral vacuum, thus making sucking impossible.

This is not only the case for breastfeeding, but also for bottle feeding using commercially available teats. It is extremely difficult for these infants to suck, and in many cases it can be impossible.

To help these infants learn to suck with some degree of success, NUK has developed a range of special teats based on scientific knowledge.* These teats are based on the specific, anatomical-morphological conditions of the alveolar region of children born with clefts - both in form and function.

The NUK Cleft Palate Teat – For children with a cleft palate.

This teat has been designed so that the large mouthpiece of the teat seals the cleft in the palate.

Important Warning: The feed hole in the teat must be manually made for each individual infant. This should be done, under medical supervision, according to the position of the cleft in the infant's palate (see fig. 1-4).

A feeding hole can be made in the teat using a hot needle. However, it is more hygienic to push a toothpick from the inside against the teat and then to cut off the tip of both toothpick and teat. Make the hole as small as possible, as you can always make it bigger!

The feeding hole should be made on the outer edge of the mouthpiece on the side furthest away from the cleft. It should not be made on the under side, as otherwise the hole will be covered by the tongue.

One of the most important characteristics of the NUK Cleft Palate Teat is that it promotes the natural sucking reflex and milking action while feeding. With every mouthful that the infant takes the large arched roof of the teat presses against the palate, which seals the cleft allowing the milk to flow into the fold of the cheek. This stops the milk from entering the respiratory tracts, preventing the infant from choking. Milk is therefore mixed with saliva, which is important for the subsequent digestion process. The NUK Cleft Palate Teat easily fits any standard neck bottle.

The NUK Cleft Lip Teat with a Medium Feed Hole (for milk)

For children with a cleft lip and those with a cleft palate where a plate has been fitted to the roof of the mouth. This teat was developed as a sucking aid for children with unilateral (one-sided) clefts and dental plates for feeding, and in exceptional cases can be used for children with bilateral clefts and palates.

These teats close the oral cavity to the atmosphere by covering the outer cleft area. This enables the infant to generate an intraoral vacuum.

The part of the teat which covers the upper lip is particularly soft. When the infant begins to suck this part seals against the upper lip preventing further air flowing through the cleft. This teat is used in cases where the cleft palate has already been sealed by a dental plate.

For children born with cleft lip and cleft palate, this teat permits the correct sucking action while feeding from the bottle, and at the same time fulfils the baby's inherent need to suck.

In contrast to other feeding methods, when using the NUK Cleft Lip Teat the otherwise frequently observed symptoms, such as choking, spitting out and even vomiting, are considerably reduced. There are also no detrimental pre-operative, orthopaedic corrective measures required.*

The NUK Cleft Lip Teat was developed in cooperation with scientists.* Form and function have been successfully tested in many hospitals worldwide. This teat is comparable with the original NUK teat and does not have the appearance or properties of a prosthesis. It has properties that are particularly important for this type of special teat, allowing it to form a seal over the cleft lip. The NUK Cleft Lip Teat easily fits any standard neck bottle.

Important Information:

- Breastfeeding is best for babies, whenever possible.
- Can be used for feeding expressed milk. Other baby feeds should only be administered in the quantity and consistency recommended by the manufacturer.
- Medical Advice: natural rubber latex products can cause an allergic reaction.
- Do not leave baby to feed alone. Avoid the use of the feeding bottle as a soother/ pacifier/ dummy as frequent and extended use of the feeding bottle with sweetened feeds, fruit juices or feeds with fruit acid can cause dental caries.
- Do not dip the teat in medicine.
- Teats and screw rings should be frequently checked for signs of wear. If damaged or showing signs of visible ageing (sticky, cracked, or hard), the teat and screw ring should be discarded.
- Teats should be stored hygienically and in a dark place, i.e. not in direct sunlight, and away from heat.

Usage:

- Feeding bottles must be cleaned and sterilised thoroughly before each use. Always remember to sterilise the accessories!
- When sterilising by boiling always ensure that there is sufficient water in the pan to prevent the bottles/teats from becoming damaged (boiling for approx. 5 minutes will suffice). When using a steam steriliser always follow the manufacturer's instructions.
- Sterilisation of teats and bottles using a dishwasher is not recommended, as the detergent can reduce the life of the product and can cause material damage.
- Bottles sealed and/or containing infant feeds should not be put in the microwave, as there is a risk of explosion and material damage. With some microwave sterilisers damage can be caused if materials with different heat absorption levels come into contact (e.g. glass absorbs higher temperatures than plastic and so the plastic may become deformed). Uneven heating also carries the risk of scalding.

Please keep address and Art. No. in case of complaint.

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