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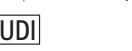
GB Prontosan®– Wound Irrigation Solution

Intended purpose: For rinsing, cleansing, moistening and decontamination of acute, chronic, and infected skin wounds and burns.

1. Introduction: Chronic skin wounds are often coated with slough, necrotic tissue and/or biofilm. These coatings are difficult to remove and lead to delayed wound healing. Prontosan® Wound Irrigation Solution is able to remove these barriers to wound healing through its cleansing activity. Acute wounds also require proper cleansing as they are generally contaminated with debris and microorganisms. These contaminants can interfere with the normal wound healing process and lead to complications such as infection. Due to the unique combination of ingredients (i.e. the antimicrobial substance, polyhexanide, and the surfactant, betaine), Prontosan® Wound Irrigation Solution is ideal for the prevention of biofilm formation. Prontosan® Wound Irrigation Solution should be used for wound cleansing prior to treatment with Prontosan® Wound Gel or Prontosan® Wound Gel X. The choice between Prontosan® Wound Gel and Gel X allows an optimal application in large surface areas as well as deep wounds. In general, both products are suitable in all conditions. However, due to the higher viscosity of Prontosan® Wound Gel X, we recommend to use it preferably for large surface area wounds including burns, whereas Prontosan® Wound Gel is more fluid and might therefore be easier to apply in deep or tunneling wounds, wound cavities and difficult to access areas. The summary of safety and clinical performance (SSCP) is available in the European database on medical devices (Euamed) and can be found using basic UDI-DI 403923900000092425 or the product name. <https://ec.europa.eu/tools/euamed>. Or requested from the Manufacturer

Prontosan® – Wound Irrigation Solution

		
Fabrikant Fabricante Fabricante Производител Üretici	Lotnummer Número de referencia Referência Каталожен номер Kataloški broj	Houdbaarheid na opening: 8 weken Caudadidad tras su abertura: 8 semanas Val. após abertura: 8 semanas Срок на годност след отваряне: 8 седмици Rok uporabe nakon otvaranja: 8 tjedna
		
Medisch hulpmiddel Dispositivo médico Dispositivo médico Медицинско изделие Medicinski proizvod	Lotnummer Lote Número do lotto Партиден номер Lot numarası	Uniek apparaat-ID Identificación única del producto Identificador único do dispositivo Партиден номер Jedinstvena identifikacija proizvoda

		
Gebruiken tot: maand en jaar Usar antes de: mes y año Val: mês e ano Да се използва до: месец и година Tarihine kadar kullanınız	Lees voor gebruik zorgvuldig de gebruiksaanwijzing Leer las instrucciones de uso Ler as instruções de uso Прочетете указанията за употреба Kullanım kılavuzunu okuyunuz	Erkende vertegenwoordiger in de EU Representante autorizado da UE Representante autorizado da UE AB Yetkilii Temsilcisi
		
Productiedatum Fecha de fabricación Data de fabrico Дата на производство Datum proizvodnje	Buiten het bereik van kinderen houden Manténgase fuera del alcance de los niños Conservar ao abrigo da vista e do alcance das crianças Да се съхранява на място недостъпно за деца Çocukların erişilebileceği yerlerden uzak tutunuz	Importeur Importador Importador Вносител Ithalatçı

		
Niet gebruiken wanneer de verpakking beschadigd is No utilizar si el envase está dañado Não usar os frascos se a embalagem estiver danificada Не употребявайте, ако опаковката е повредена Nemojte upotrebljavati ako je pakiranje oštećeno	Steriel barrièresysteem met beschermende verpakking voor eenmalig gebruik Sistema de barrera estéril único con embalaje protector en el exterior Sistema de barreira estéril único com embalagem exterior de proteção Единична стерилна преградна система със защитна външна опаковка Sustav jednostruke sterilne barijere s vanjskim zaštitnim pakiranjem	Eén patiënt – meervoudig gebruik Un solo paciente – Uso múltiple Paciente único – várias utilizações Един пациент – Многостратна употреба Tek Hasta - Çoklu Kullanım
		
Flessen tegen direct zonlicht beschermen Protector de la luz solar directa Protetores da luz solar direta Бутилките да се пазят от пряка слънчева светлина Şişelerin doğrudan güneş ışığına maruz bırakılmayın	Enkelvoudig steriel barrièresysteem Sistema de barrera estéril único Sistema de barreira estéril único Единична стерилна преградна система Sustav jednostruke sterilne barijere	Bevat een geneesmiddel Contiene un fármaco Contém substâncias medicas Сдържа лекарствено вещество Sadržaj lijekovitu tvar

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