# Holthaus Medical

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Aqua, Alcohol, Cellulose, Polyester, Propylenglycol, Glycerol, Nonaethylene Glycol Monododecyl Ether, Formic acid, PVP, 2-isopropyl-5-methylphenol

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## Wound Cleansing Tissues

### Intended Use

For external cleaning of small cuts, bruises, chafing and stab wounds.

### Product description

Contaminations, which enter the wound, are the main reason for wound healing problems. Mechanical removal of these contaminations immediately after injury is therefore a suitable means to promote wound healing and to prevent infections.

### Application

1. Remove the wipe from the packaging and unfold it.
2. Carefully clean the wound superficially (for example, by dabbing or stroking away from the wound).

### Notes

Use the product only for the initial treatment of superficial wounds. If the depth of the wound is cannot be determined, the wound is heavily contaminated, or bleeding profusely, consult a physician.

Check your tetanus vaccination status, as there is a risk of tetanus infection from any wound. If in doubt, contact your physician.

If the wound does not heal and, for instance, shows signs of inflammation (e.g. redness), contact your physician – the product is not suitable for regular wound care during the healing process.

The product is suitable for single use only. Properly dispose of the wipe after use and do not reuse it.

If the packaging of the product is damaged, do not use the product.

Current experiences with pregnant and nursing women using this product do not indicate a hazard. Accumulation of the fluid in breast milk is unlikely, as there is no distribution in the body due to the short contact time with the wound cleansing wipe.

### Contraindications

As a precaution, do not use on babies and infants under three years of age, likewise not on persons who have a proven allergy (contact sensitization) against one of the ingredients specified above.

### Storage

Keep product out of reach of children.

Do not use the product after the expiration date.