BSS™

STERILE IRRIGATING SOLUTION

ENGLISH.....	2-3
ČEŠTINA.....	4-5
DANSK	6-7
NEDERLANDS	8-9
SUOMI.....	10-11
FRENCH	12-13
DEUTSCH	14-15
GREEK	16-17
ITALIAN	18-19
NORSK.....	20-21
POLISH	22-23
PORTUGUÊS.....	24-25
ROMANIAN	26-27
ESPAÑOL.....	28-29
SVENSKA.....	30-31

STERILE IRRIGATING SOLUTION**DESCRIPTION**

BSS™ Sterile Irrigating Solution is a sterile, physiologically balanced salt solution for extraocular and intraocular irrigation. The function of BSS™ Sterile Irrigating Solution is to rinse and moisten ocular tissues during various surgical procedures of the eyes. BSS™ Sterile Irrigating Solution has no pharmacological activity. The qualitative and quantitative electrolyte composition, pH, and osmolality approximate that of human aqueous humor.

COMPOSITION

BSS™ Sterile Irrigating Solution is a sterile physiological salt solution, each mL containing Sodium Chloride (NaCl) 0.64%, Potassium Chloride (KCl) 0.075%, Calcium Chloride Dihydrate ($\text{CaCl}_2 \cdot 2\text{H}_2\text{O}$) 0.048%, Magnesium Chloride Hexahydrate ($\text{MgCl}_2 \cdot 6\text{H}_2\text{O}$) 0.03%, Sodium Acetate Trihydrate ($\text{C}_2\text{H}_3\text{NaO}_2 \cdot 3\text{H}_2\text{O}$) 0.39%, Sodium Citrate Dihydrate ($\text{C}_6\text{H}_5\text{Na}_2\text{O}_7 \cdot 2\text{H}_2\text{O}$) 0.17%, Sodium Hydroxide and/or Hydrochloric Acid (to adjust pH), and Water for Injection. BSS™ Sterile Irrigating Solution is an essentially unbuffered solution. Acid Buffer Capacity = 0.0014 M/L/pH; Base Buffer Capacity = 0.00019 M/L/pH.

INTENDED USE

BSS™ Sterile Irrigating Solution is an isotonic solution for use by a trained ophthalmic surgeon in irrigating tissues of the eyes in patients undergoing ophthalmic surgery.

INDICATIONS AND USAGE

For use as an extraocular and intraocular irrigating solution.

CONTRAINDICATIONS

There are no known contraindications with the use of BSS™ Sterile Irrigating Solution when used as recommended.

WARNINGS

1. This product is for IRRIGATION during ophthalmic surgery only. DO NOT use for injection or intravenous infusion.
2. The safety and effectiveness of BSS™ Sterile Irrigating Solution combined with additives have not been substantiated by Alcon.
3. DO NOT use unless product is clear, seal is intact, and container is undamaged.
4. DO NOT use if sterile barrier is compromised or unintentionally opened before use.
5. DO NOT use if product is discolored or contains a precipitate.
6. DO NOT re-use this product. This product is for SINGLE patient use only and should not be used in more than one patient. Reuse of this single-use device may result in serious injury, such as but not limited to infection.
7. This solution contains no preservative; unused contents should be discarded.

PRECAUTIONS

1. Open under aseptic conditions only.
2. Prior to use, check the following: tip should be firmly in place and irrigating needle should be properly seated. Squeeze out several drops before inserting into anterior chamber.

3. The needle should be removed from the anterior chamber prior to releasing pressure to prevent suction.
4. Studies suggest that intraocular irrigating solutions which are iso-osmotic with normal aqueous fluids should be used with caution in diabetic patients undergoing vitrectomy since intraoperative lens changes have been observed.
5. There have been reports of corneal clouding or edema following ocular surgery in which BSS™ Sterile Irrigating Solution was used as an irrigating solution.

PERFORMANCE CHARACTERISTICS

A clinical evaluation of published market experience data, clinical investigations, and clinical literature data support that BSS™ Sterile Irrigating Solution has an acceptable safety and performance profile when acting as an intraocular irrigating solution during ophthalmic surgery. The data show that BSS™ Sterile Irrigating Solution is a suitable intraocular irrigating solution for a variety of surgical procedures, including anterior and posterior segment procedures, in both adult and pediatric populations. Adverse events with BSS™ Sterile Irrigating Solution occurred at a low rate, were mostly non-serious, and were expected when using an irrigating solution during intraocular surgery.

CLINICAL BENEFITS

Based on a clinical evaluation of BSS™ Sterile Irrigating Solution, the clinical benefits of BSS™ Sterile Intraocular Irrigating Solution in intraocular surgery include:

- Irrigating and moistening of ocular tissue
- Providing a protective barrier of ocular tissues during surgery
- Rinsing away surgical debris
- Maintains anatomical and physiological integrity of the eye

ADVERSE REACTIONS

Irrigation or any trauma to the corneal endothelium may result in corneal swelling or bullous keratopathy. Postoperative inflammatory reactions as well as incidents of corneal edema, increased intraocular pressure, and corneal decompensation have been reported.

SERIOUS INCIDENT REPORTING

Any serious incidents that may reasonably be regarded as device related should be reported to Alcon Laboratories, Inc.:

By Phone: Contact the local country

office or your Alcon distributor.

By Email: qa.complaints@alcon.com

Website: <https://www.alcon.com/contact-us/>

Each device is identified by batch code which provides traceability, and this information should be given to Alcon.

NOTE: These serious incidents must also be reported to the competent authority for medical devices of the appropriate State.

DOSAGE AND ADMINISTRATION

The adapter plug is designed to accept an irrigating needle. Tissues may be irrigated by attaching the needle to the dispensing bottle as explained below. External irrigation may be done without the irrigating needle.

Method of using Adapter Plug for LUER-LOK® Hub Ophthalmic Irrigating Needle:

1. Aseptically remove dispensing bottle from blister by peeling paper backing.

2. Snap on surgeon's sterile irrigation needle. Push until firmly in place and twist slightly.
3. Test assembly for proper function before use (See Precaution 2).

No special disposal of the BSS™ Sterile Irrigating Solution is required after use.

HOW SUPPLIED

BSS™ Sterile Irrigating Solution is provided in 15 mL dispensing bottles sterilized by steam. Sterility is guaranteed until the expiration date unless the sterile barrier is damaged or opened.

STORAGE

Store BSS™ Sterile Irrigating Solution between 2°C to 30°C.

SYMBOLS USED ON LABELING

SYMBOL	DEFINITION
	Medical device
	Caution
	Consult instructions for use
	Electronic instructions for use
	Not for intravenous use
	Do not re-use
	Temperature limit
	Sterilized using steam
	Single sterile barrier system
	Use-by date
	Manufacturer
	Date of manufacture
	Batch code
	Sterile Irrigating Solution
	Authorised Representative in the European Community
	Do not use if sterile barrier is compromised

DIS

Distributor

Alcon Laboratories, Inc.
6201 South Freeway
Fort Worth, Texas 76134-2099, USA

EC REP

Alcon Laboratories Belgium
Lichterveld 3
2870 Puurs-Sint-Amands
Belgium

Distributed in Australia by:
Alcon Laboratories (Australia) Pty Ltd
15 Talavera Road
Macquarie Park NSW 2113 Australia
Phone Toll Free: 1800 224 153

Distributed in New Zealand by:
Alcon Laboratories (New Zealand) Limited
12 St Marks Road
Remuera, Auckland 1050 New Zealand
Free Phone: 0800 101 106

*LUER-LOK is a trademark of Becton, Dickenson & Co.

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© 2020 Alcon Inc.
*LUER-LOK trademark of Becton, Dickinson and Company.
Alcon Nordic A/S