DE octenilin® Wundspüllösung

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Opening of octenilin® Wound Irrigation Solution:

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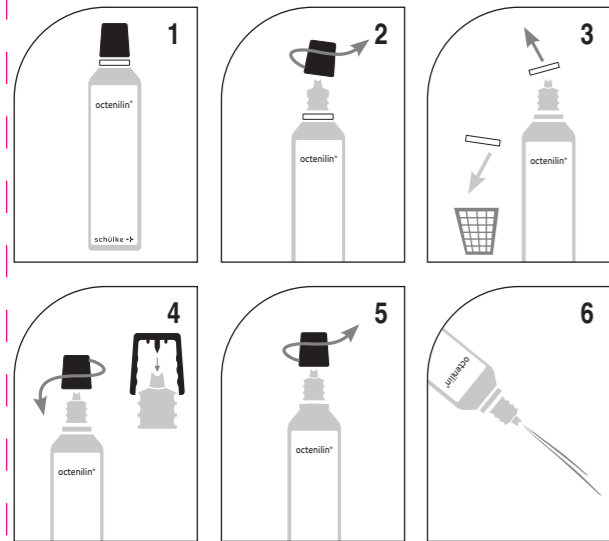
EN octenilin® Wound Irrigation Solution

Instructions for use

1. Introductory remarks: Chronic skin wounds are almost always covered with exudate, a thick sequestered fibrin layer, necrotic tissue and cellular debris. This covering provides good conditions for the growth not only of pathogens, but also of transient or resident pathogens of the skin flora, considerably delaying wound healing. Careful wound cleansing and wetting of the wound with octenilin® Wound Irrigation Solution is therefore indicated. Due to its low surface tension, octenilin® Wound Irrigation Solution has excellent wetting properties and thus cleansing properties on tissue surfaces, even difficult to reach surfaces, such as fissures, and wound pockets.

2. Product profile and applications: octenilin® Wound Irrigation Solution can be used, e.g.

- for cleansing and moistening of chronic skin wounds
- for tissue-sparing wetting of bandages and wound coverings such as compresses, gauze, cushions, sponges, gels, hydrofibres, alginates, hydrocolloids etc.
- when changing bandages, to free encrusted bandaging material or other encrusted wound coverings
- cleansing of ports for PEG/PEJ tubes or drainage tubes.
- cleansing peristomal skin areas
- combined with instillation in negative pressure wound therapy

The use of octenilin® Wound Irrigation Solution causes no pain and is also well tolerated by persons with allergies. Unpleasant odours are quickly removed

3. Application in general: *Only for external use on skin wounds.* octenilin® wound irrigation solution is suitable for repeated and long term use. The long term use of octenilin® wound irrigation solution should be done under medical supervision. Wounds should always be irrigated and cleansed with octenilin® Wound Irrigation Solution to remove easily soluble encrustations before further treatment with octenilin® Wound Gel (see separate instructions for use).

Bandages or dressings soaked with octenilin® Wound Irrigation Solution can be used for cleansing. Bandages are frequently encrusted and stick to wound surfaces. If these bandages are removed when dry, it may cause further damage in the wound area, further delaying healing. In cases where bandages are difficult to remove, it is advisable to wet or soak them with octenilin® Wound Irrigation Solution until it is possible to gently remove them without traumatising the wound surface.

Application should be repeated frequently until all encrustations or necrotic tissue are easily removed and the wound looks clean. This is a good precondition for natural wound healing and ensures optimal visualisation for surgical debridement.

Patients with chronic wounds are often temperature sensitive. octenilin® Irrigation Solution can be warmed to body temperature *immediately before use.*

4. Cleaning around the wound: Thorough extensive cleaning of the entire area surrounding the wound with octenilin® Wound Irrigation Solution is recommended to minimise the risk of transmitting pathogens to the wound.

5. Tissue tolerability and biocompatibility: non-irritating, non-sensitising, pain-free, without tissue toxicity, no impairment of granulation and epithelialisation. The good tolerability of Octenidine has been confirmed by clinical experience over many years and by clinical studies also on chronic wounds.

6. Side effects: none observed to date.

Any serious incidents related to the product are to be reported to the manufacturer and competent authority.

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schülke -†

7. Interactions: Do not use in combination with anionic surfactants or other wound cleansing soaps, ointments, oils, enzymes etc. since this may adversely affect the preservative. Do not use in combination with PVP-iodine, since this may cause discoloration and impair the antiseptic effect of the PVP-iodine.

8. Contraindications: octenilin® Wound Irrigation Solution should not be used in patients with known or suspected allergy to one or more of the ingredients. If in doubt consult a physician.

octenilin® Wound Irrigation Solution should not be used on hyaline cartilage, the eyes, ears, nose, urinary bladder and in the abdominal cavity! Do not use for infusion or injection! Must not be taken by mouth.

To prevent possible tissue injury, the product must not be injected or applied to tissues with pressure. Adequate drainage from wound cavities must be provided (e.g. flexible drain tube).

9. General safety instructions: Only to be applied to the wound externally.

Only flawless and undamaged container must be used.

Use only intact and undamaged packs. Protect the product against exposure to direct sunlight.

To prevent the introduction of bacteria when using octenilin® Wound Irrigation Solution, ensure that the packaging does not come into contact with the wound.

Keep out of the reach and sight of children!

10. Composition/ technical data: Aqua valde purificata, Glycerin, Ethylhexylglycerin, Octenidine HCl.

Appearance and odour: clear, colourless and nearly odourless aqueous solution.

Stability at room temperature, see stability information.

Original condition: **STERILE A** – sterile, sealed.

Conservation and duration of use: octenilin® Wound Irrigation Solution is preserved in such a way that the risk of spreading pathogens to surrounding tissue is markedly reduced when it is used.

Duration of use: 8 weeks after the bottle has been opened

Although sterility is compromised once the bottle has been opened due to the manner of conservation an 8-week period of use is possible as long as the bottle is closed immediately after each time it is used and the bottle opening is protected against contamination or direct contact. Bottles that have been in contact with the wound must be discarded.

Disposal: Fully emptied packs can be disposed of as household waste.

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