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EN**A SIMPLE, EFFECTIVE EAR CLEANING METHOD****AUDISPRAY ADULT - 100% natural**

AUDISPRAY ADULT contains a microbiologically controlled hypertonic seawater solution. AUDISPRAY ADULT is an alternative to cotton buds.

What is AUDISPRAY ADULT for?

AUDISPRAY ADULT softens and dissolves wax, thus facilitating its natural removal. Used regularly, AUDISPRAY ADULT prevents the build-up of wax and impurities in the ear canal. AUDISPRAY ADULT, with no gas propulsion and fitted with an anti-surge nozzle, is ideal for regular and effective ear hygiene.

Who can use AUDISPRAY ADULT?

Ages 12 and up: 1 to 2 sprays twice per week. **Hearing Aid Wearers:** The use of Audispray every night after removing the devices is recommended to avoid possible malfunctions due to the presence of wax.

How do I use AUDISPRAY ADULT?

- Point the nozzle towards the ear and spray 1 to 2 times per ear, keeping the bottle vertical.
- Massage the base of the ear for a few seconds.
- Tilt your head to one side to remove excess spray and dry.
- Rinse the nozzle in hot water after each use.

Contraindications

Should not be used in the event of an ear infection or in general on any person with a past or present medical history (dizziness, medical or surgical procedures, etc.)

Precautions for use

Consult the pharmacist or your doctor in case of doubt or pain before using the product. In the event of an existing wax impaction, AUDISPRAY ADULT may lead to the swelling of the impaction, causing discomfort. If this occurs, consult your doctor.

It is recommended to use AUDISPRAY ADULT within one year of first using the bottle. Do not use after the expiry date shown on the packaging and on the bottle.

Any serious incident connected to the use of AUDISPRAY ADULT should be reported to the manufacturer and to the competent authority of the Member State where the user lives.

AUDISPRAY ADULT is a medical device complying with EU Directive 93/42/EEC

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