**Product Description:**

The BARIKOS KS eye wash bottle is designed for immediate eye irrigation to help prevent eye injuries. It allows for cleaning of the eye and helps to flush out foreign bodies or harmful substances. Whether in chemical laboratories, manufacturing industries, or forestry – it is always recommended to have an eye wash bottle readily accessible.

**Intended Use:**

This medical device is intended for cleaning the eyes by flushing out foreign bodies, dust, chemicals, or other harmful substances. No knowledge of the substance is required, as the solution is pH-neutral and water-based (suitable for both acids and alkalis).

**Instructions for Use:**

- » Remove the protective cap
- » Gently rinse the affected eye
  - While standing/sitting: By squeezing the bottle walls, the rinsing solution is pushed up through the riser tube and reaches the affected eye via the spray head. The fluid collects in the ergonomically shaped eye cup and is directed out through the side drainage channels.
  - While lying down: The rinsing solution flows out through the open tilting valve.

**Important Information:**

- » This product is intended for single use only.
- » The eye wash bottle must be placed in a clearly visible, easily accessible location close to the hazard area.
- » Durable water (gamma-irradiated).
- » No risk of contamination as often found with fixed eye wash stations.
- » Gentle rinsing due to the ergonomically shaped eye cup and the wide, fan-shaped spray.
- » Complies with DIN EN 15154-4 standard for safety showers.
- » CE marking in accordance with the EU Medical Device Regulation (MDR)
- » Does not replace examination by an eye specialist!
- » BARIKOS KS Eye Wash Bottle – effective volume: 620 ml
- » Mini BARIKOS KS Eye Wash Bottle – effective volume: 175 ml
- » In emergencies, the bottle can be refilled for extended rinsing.

**Storage:**

- » Store at a temperature between +15 °C and +37 °C (ideally between +20 °C and +25 °C).
- » Protect from extreme cold and heat.
- » Do not expose to direct sunlight.

**Placement:**

According to EN 15154-4, the following must be observed:

- » Instructions for mounting the eye wash bottle in a clearly visible and easily accessible location, within reach and in close proximity to the hazard area.
- » A safety sign in accordance with ISO 3864-1 must be installed near or on the eye wash bottle to indicate its location.
- » A water-resistant maintenance label must be affixed near or on the eye wash bottle, documenting maintenance in accordance with section A.4.

**Shelf Life:**

The product's expiration date is printed on the eye wash bottle. Do not use the bottle after this date has passed. When sealed, the product has a shelf life of 2 years.

**Disposal:**

Disposal must comply with regional, national, and local regulations. We recommend contacting the relevant authorities or an authorized waste disposal company for appropriate waste handling information.

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**Instructions for Use – Eye Wash Bottle**  
Eye Wash Bottles BARIKOS KS and Mini BARIKOS KS

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