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Medical device
– a pure mineral gel

Intended use

Herpes Gel is used for the external, physical and supportive treatment of herpes blisters.

Product properties

Herpes Gel does not contain any greasy ingredients that favour bacteria and viruses. The formula is based on a mineral salt solution that breaks down into its natural components after application. The medical product supports and accelerates the healing of the blisters if it is applied in time at the first signs such as tingling and burning. Applying the herpes gel creates a protective film. The silicate contained in the gel absorbs exudates like a blotting paper due to its extraordinary binding capacity. It thus contributes to a reduction of the open and weeping areas. The spread of infection-promoting germs is inhibited and healing is supported. Herpes Gel has been rated as dermatologically "very well" tolerated and is clinically proven. Herpes Gel is also suitable for allergy sufferers and children.

Instructions for use

Shake well before use! Clean hands thoroughly before and after use. Depending on your needs, Herpes Gel can be applied thinly to the affected areas several times a day. Cotton swabs or the cleaned fingertip are suitable for this purpose. For hygienic reasons, the tip of the tube should not come into contact with the blisters. Close the tube again after use. The typical duration of use is 4 to 5 days. The product should be used until the herpes blisters are completely crusted or healed, but not longer than 10 days. There are no known undesirable side effects. A slight tingling sensation during application, which is often perceived as pleasant, is a good indication of the physical effect of the product. Accidental swallowing of small amounts when applied to the lips is not a problem.

Contraindications / restrictions of use

For external use in case of cold sores. Herpes Gel should not be used in cases of known allergies to any of the ingredients. In children under 12 years of age, the application should be supervised by an adult. For infants aged 0-6 years, use should always be after consultation with a doctor. For use during pregnancy, while breast-feeding and in people with severe immunodeficiency, consult your doctor or pharmacist beforehand. If your symptoms worsen or do not improve after 5 days, you should always consult a doctor.

Safety information

Avoid eye contact. In case of eye contact, rinse with plenty of water. Use the product only until the stated expiry date and within 3 months after opening. Please report any potentially serious incident involving the product immediately to Promtomed (vigilanz@promtomed.de) and the competent authority in your country.

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Prontomed GmbH