filme Nasale

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filme Nasale

EN Oil for the nasal mucosa
Nasal care pump contains no gas

20 ml / 0.70 FL. OZ. bottle

COMPOSITION

Tocopherol acetate (20mg / dispensed), Medium-chain triglycerides.

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EXCLUSIVE SALES LICENSEE FOR ITALY

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WHEN SHOULD FILME NASALE BE USED

FILME NASALE is Tocopherol Acetate (oil base Vitamin E), which forms a protective film. It helps to restore the normal hydration of the nasal surface in individuals with discomfort such as burning, itching, reddening, irritation or nasal discharge resulting from insufficient or absent nasal secretion, induced by external factors: such as physical stress, chemicals, environmental, menopause, following surgical intervention, topical medication or nasal wash. Indicated for the hydration of the nasal mucosa in subjects with atrophic rhinitis. It creates a protective barrier on the nasal mucosa that can assist to alleviate the symptoms of allergic rhinitis.

FREQUENCY AND WHEN THE PRODUCT IS TO BE APPLIED

Pump 2 squirts into each nostril at least twice a day to restore normal hydration of the mucosa. In case of atrophic rhinitis, dispense 2 squirts into each nostril 3 times a day for at least 3 months to promote trophism of the nasal mucosa. During seasonal allergic rhinitis in particular when allergens are present, 2 squirts in each nostril at least 3 times a day as often as required (up to 6 times a day).

CONTRAINDICATIONS

Hypersensitive towards the components.

WARNING

Do not inhale when dispensing. No adequate clinical data exist on the use in children and adolescents. It is recommended to consult your doctor before use.

KEEP OUT OF REACH OF CHILDREN.

INTERACTION WITH MEDICINES AND OTHER INTERACTIONS

There are no known interactions with other medicines or products used on nasal mucosa. In the case of application of more than one product leave an interval of at least 5 minutes between each product.

HOW TO USE THIS DEVICE

1. Remove protective cap
2. Before using the first time, prime pump by depressing 5 times
3. Place tip of nozzle just past nasal opening, in a vertical position
4. While inside nasal opening, slightly angle nozzle outward and pump
5. Repeat the process in the other nostril

METHOD AND ROUTE OF ADMINISTERING

For topical nasal use.

SIDE EFFECTS

Although FILME NASALE is extremely tolerable, in some rare cases it may cause a stinging effect, irritation, allergic reaction, reddening and swelling in the area of application. If this occurs discontinue use and consult a doctor. If any of the side effects gets serious, or if you notice any side effects not listed in this leaflet, please tell your doctor or pharmacist.

STORAGE OF THE MEDICAL DEVICE

DO NOT USE AFTER THE INDICATED EXPIRY DATE ON THE INNER PACKAGING OR ON THE OUTER PACKAGING. STORE AT ROOM TEMPERATURE.

MEDICAL DEVICE.

AFTER USE, DISPOSE OF IN ACCORDANCE WITH YOUR NATIONAL LEGISLATION.

DO NOT DISPERSE IN THE ENVIRONMENT.

European patent

CE 0051

CONTAINS NO AROMA
NO COLOURANT
NO PRESERVATIVES
NO ESSENTIAL OILS
Nickel (< 1 ppm) and Gluten (< 10 ppm) tested

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