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## Instructions for use – KerraSol™ wound spray

### Product properties:

KerraSol™ wound spray is used for cleaning, moistening, antimicrobial decontamination and infection prevention in the wound.

The application of KerraSol™ wound spray is non-irritating. KerraSol™ wound spray is used for cleaning the wound by helping to remove biofilms and fibrinoid residues. In addition KerraSol™ wound spray can aid the gentle removal of dressings.

The preservative (sodium hypochlorite) contained in KerraSol™ wound spray decontaminates a broad spectrum of bacteria (incl. pseudomonas aeruginosa, MRSA/MRE), viruses and fungi.

Wound odour is quickly and reliably eliminated when using KerraSol™ wound spray.

### Application:

The wound should first be cleaned to remove dirt, blood and any residues. The affected area is sprayed with KerraSol™ wound spray by pressing the pump on the bottle several times, at a short distance to fully moisten the wound. If necessary, for mechanical cleaning and debridement, in the event of major contamination, the application can be repeated several times. Once the cleaning procedure is finished, a secondary cleaning solution is not required (i.e. any residual KerraSol™ wound spray can be left in the wound).

Dressings or bandages that are difficult to remove are sprayed with KerraSol™ wound spray until they are wet. After a short waiting period, they can then be carefully removed virtually pain-free. If needed, KerraSol™ wound gel or KerraSol™ wound spray gel can be applied. There are no restrictions regarding the frequency and duration of use. A slight chlorine odour indicates the product is working.

### Application restrictions:

There are no known side effects of using KerraSol™ wound spray. KerraSol™ wound spray should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Interactions with other substances are not expected. During pregnancy and when breastfeeding KerraSol™ wound spray should be used with caution. Not intended for infusion or injection!

### General safety instructions:

Only apply to external dermal wounds. Do not swallow or spray in eyes. Keep away from children. Only use intact, undamaged bottles.

Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks.

### Ingredients:

Water, sodium hypochlorite <0.08% (electrochemically activated mineral saline solution)

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Made in Germany

Distributed by:  
Crawford Healthcare GmbH  
Medizinpark 1  
83626 Valley / Oberlaindern  
Tel: +49 (0)8024 46868-20  
info@crawfordhealthcare.com  
www.crawfordhealthcare.com

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Prontomed GmbH