Mölnlycke® Granudacyn®


Wound Irrigation Solution
&
Wound Irrigation Solution NPWT

Manufacturer / Fabricant:
P.G.F. Industry Solutions GmbH
Katzmoosstr. 26a
5161 Elixhausen
AUSTRIA

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Temperature limit
Température limite

Granudacyn® Wound Irrigation solution

en

Product description

Granudacyn® Wound Irrigation solution is a ready-to-use, wound irrigation solution for the cleaning and irrigation of acute, chronic and contaminated wounds.

Cleaning of wounds prevents wound infection. In wounds that are already infected, adjuvant wound irrigation can help to significantly reduce the microbial load. This results from the mechanical effect of the irrigation, thus promoting wound healing.

Content: Water, sodium chloride, hypochlorous acid, sodium hypochlorite

Indications for use

Granudacyn® Wound Irrigation solution can be used for the thorough cleaning of:

- all chronic wounds of any depth, such as: diabetic foot ulcer, all forms of leg ulcer, pressure sores
- all acute wounds such as cuts, bites, lacerations, burst wounds and abrasions
- surgical wounds (intraoperative and postoperative)
- wounds with exposed cartilage, tendons, ligaments or bone
- 1st and 2nd degree burns
- radiation ulcers
- fistulas and abscesses
- body cavities such as the mouth, nose and ears
- critically colonized or infected wounds
- soft tissue injuries

Granudacyn® Wound Irrigation solution can be used for instillation in negative pressure wound therapy (NPWT).

Precaution(s)

- Granudacyn® Wound Irrigation solution is not suitable for systemic use, i.e. Granudacyn® Wound Irrigation solution must not be administered by intravenous or intra-arterial injections or infusions.
- Should not be combined with other wound cleaning products (such as enzymatic wound cleaners)
- Do not mix with other products or chemicals
- Only use intact and undamaged packages
- Do not use Granudacyn® Wound Irrigation solution on patient and/or user with known hypersensitivity to the product materials/components.
- Oral ingestion, although harmless, is nevertheless not recommended.

Instructions for use

Granudacyn® Wound Irrigation solution does not need to be rinsed or neutralised.

The wound should be irrigated thoroughly with Granudacyn® Wound Irrigation solution.

To enhance the cleaning effect, it is recommended that the solution be allowed to soak in for 60 seconds to dissolve any encrustations.

This procedure can be repeated several times a day or at each dressing change.

Alternatively, the wound may be cleaned with gauze soaked in Granudacyn® Wound Irrigation solution.

Encrusted wound dressings are easily loosened with Granudacyn® Wound Irrigation solution and without trauma. To support the mechanical cleaning of the wound, the application time of Granudacyn® Wound Irrigation solution can be extended to 15 minutes to remove stubborn encrustations.

Granudacyn® Wound Irrigation solution can be used for up to 60 days once opened. The date on which it is first opened is to be written in the space provided on the bottle.

Granudacyn® Wound Irrigation solution may be warmed to up to 37°C before use.

Granudacyn® Wound Irrigation solution can be used by lay persons under supervision of health care professionals and is suitable for repeated use.

Granudacyn® NPWT should only be used in compliance with the instructions for use supplied by the NPWT system manufacturer by qualified health care professionals.

Granudacyn® Wound Irrigation solution can be used with Mölnlycke Health Care foam dressings: Mepilex, Mepilex Border and Mepilex Border Flex.

Special storage conditions and handling conditions

Store the product between 3 and 25°C.

Protect from direct sunlight.

Disposal should be handled according to local environmental procedures.

Other information

Granudacyn® is registered trademark of Mölnlycke Health Care AB.

Any serious incident that occurs from the use of Granudacyn® Wound Irrigation solution should be reported to the manufacturer and the competent authority in your country.

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Mölnlycke Health Care Pty. Ltd.