ocuers wipes

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INHALT

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ocuers wipes

Single-use, non-woven ophthalmic gauze soaked in a solution with Hyaluronic acid and Aloe Vera with a cleansing, moisturising, soothing and emollient action around the eyes and on the eyelid surface.

INDICATIONS

- Cleansing, moisturising, soothing and emollient action.
- Ideal in mitigation of irritative phenomena affecting the conjunctiva.
- Ideal to give relief in case of skin desquamation resulting from inflammatory or allergic phenomena of the eyelids.
- Ideal as adjuvant in the treatment of blepharitis and blepharon-conjunctivitis.
- Useful for removing residues of cosmetic products from the eyelids and eyelashes.
- Compatible with contact lenses.

INSTRUCTIONS FOR USE

- Wash your hands thoroughly.
- Open the sachet starting at the pre-cut notch and remove the gauze.
- Close your eye and apply the gauze to the eyelid area, massaging gently to ensure proper hygiene.
- Use a new gauze for each eye.
- Discard the gauze after use.
- Use the gauze one or more times a day or as advised by your specialist.
- To increase the effectiveness of the mechanical removal of eye secretions, directly immerse the sachet in lukewarm water at a temperature not exceeding 40°C for 10 minutes.
- To increase eye relief, cool the sachet in the refrigerator at a temperature not below 5°C for 10 minutes.
- Dispose according to the local laws.

In the presence of secretion and of any residues of cosmetic products on the eyelids and eyelashes, it is preferable to soften these before their removal by making gentle outward movements.

WARNINGS

- Single-use medical device for external use.
- Avoid direct contact with the inside of the eye.
- In the event of adverse reactions, stop using the product and contact a specialist.
- Do not use in case of allergy or hypersensitivity to any of the product components.
- Keep away from heat sources and store in a dry place at a temperature between 5°C and 35°C.
- Keep out of the reach of children.
- Do not use the product after the expiry date.
- Never reuse the same gauze, but discard it immediately after use.
- Never apply the same gauze to both eyes, but always use a new gauze for each application.
- Do not use the product if the package looks damaged.
- The slightly yellow colour of the gauze is characteristic of the product.

COMPOSITION

Non-woven gauze soaked in a cleansing solution with Hyaluronic acid and Aloe vera gel.

Does not contain any component from animal origin.

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Imp. CH: Contopharma AG
Decomplex AG
innomedis AG medical care