VagisanCare Creme-Gleitgel

VagisanCare

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VagisanCare Cream-Lubricant

VagisanCare

VagisanCare is your series of products for the daily care of the female intimate area. Behind these products is a scientific team that develops and manufactures products for the female intimate area in order to offer you effective support in your daily intimate care routine: By using VagisanCare, you can do your sensitive intimate area a favour, as well as you feel at ease and cared-for every day.

Quality is always our top priority. That is why all VagisanCare products are subject to strict quality controls. They only contain selected ingredients of the highest quality and are made in Germany.

You will find further information at www.vagisancare.com. Please contact us there if you have questions regarding VagisanCare. We look forward to your visit.

VagisanCare Cream-Lubricant – Package leaflet –

Dear user, please read the following package leaflet carefully because it contains important information that you need to know when using this medical device. Keep this leaflet. You may need to read it again.

Application area

VagisanCare Cream-Lubricant is used to improve lubrication during intercourse if the vagina is too dry. It can also be used during pregnancy and breast-feeding.

A feeling of dryness during intercourse can occur, among other things,

- due to stress
- due to the use of hormonal contraceptives (e.g. the pill)
- after vaginal, uterine or ovarian surgery
- due to a drop in sexual hormone production in women during the menopause

In the case of dryness in the vaginal entrance and the vagina, the lubricating ability during intercourse is often impaired in such a way that it may result in pain and cramping.

Product information

VagisanCare Cream-Lubricant improves lubrication during intercourse and ensures a pleasant feeling during sex. It contains smooth lipids and does not dry out during sex. Thanks to its lasting lubricating effect, it facilitates intercourse and helps to relieve pain caused by dryness.

The Cream-Lubricant is very well tolerated and does not have any effect on sperm vitality, therefore it can also be used if you wish to conceive. It can easily be washed off and does not leave any stains on clothing.

Application

Prior to intercourse, use the fingers to apply a thin layer of VagisanCare Cream-Lubricant to the vaginal entrance or directly on the penis.

Possible side effects

In isolated cases, temporary mild intolerance reactions may occur (such as itching, burning, redness).

Restrictions for use

VagisanCare Cream-Lubricant must not be used in the case of a known hypersensitivity or allergy to any of the ingredients.

VagisanCare Cream-Lubricant must not be used together with condoms made of latex, polyisoprene or polyurethane since the tensile strength and leak tightness of these condoms can be reduced and their protective effect is no longer guaranteed. The protective effect of diaphragms can also be impaired by the Cream-Lubricant. The Cream-Lubricant should not be used immediately after giving birth.

The Cream-Lubricant should also not be used in case of vaginal infections (e.g. vaginal candidiasis); the doctor should be consulted in such cases.

If the tube is damaged or if there are any changes in the appearance of the gel, do not use the product and contact us using the information under „Contact in the event of product queries“ below.

Information

VagisanCare Cream-Lubricant is not a contraceptive and does not have any contraceptive properties. It is designed to improve lubrication during intercourse. VagisanCare Cream-Lubricant is not suitable for the lubrication of urinary catheters, other catheters, tubes or probes; the product has not been tested for these purposes. It must also not be used as an ultrasound gel as it is a lipid-containing gel.

Ingredients

Paraffinum Liquidum, Polyethylene, Caprylic/Capric Triglyceride, Laureth-4, Glycerol Oleate, PEG-20 Glycerol Triisostearate.

Storage and shelf life

VagisanCare Cream-Lubricant must be kept out of the sight and reach of children.

Keep the tube tightly closed and do not store above 25 °C.

The expiry date is printed on the flap of the outer packaging and/or on the crimp at the end of the tube. Do not use the Cream-Lubricant after this date!

Pack sizes

Tube in folding box with 10 g and 50 g gel.

Manufacturer and distributor

Dr. August Wolff GmbH & Co. KG
Arzneimittel
Sudbrackstrasse 56
33611 Bielefeld, Germany

Contact in the event of product queries

Tel.: +49 521 8808-05
Fax: +49 521 8808-334
E-mail: aw-info@drwolffgroup.com

Medical device

CE 0483

Date of revision of the text
January 2020

**Do you already know
the other products from
VagisanCare?**

**VagisanCare
Shaving-Balm**
soothes & cares

**VagisanCare
Shower-Foam**
refreshes & preserves

Meaning of the symbols on the carton and tube:

		 25 °C		
Batch number	Best before	Do not store above 25 °C	Follow the instructions for use	CE marking

Vagisan – For a healthy intimate area

In addition to daily care, there are times when your sensitive intimate area needs special support. The experts from Vagisan develop cosmetic and medical products that provide you with effective support for a healthy intimate area in special situations. You can find more information at www.vagisan.com.