marimer

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- Wang J. et al., «Efficacy of buffered hypertonic seawater in different phenotypes of chronic rhinosinusitis with nasal polyps after endoscopic sinus surgery: A randomized double-blind study.» Am. J. Otolaryngol., vol. 41, pp. 1-8, 2020.
- Çalgılı et al., «Efficacy of hypertonic and isotonic seawater solutions in chronic rhinosinusitis.» Med. Glas., vol. 7, no. 2, pp. 116-123, 2010.
- Friedman M. et al., «Dead Sea Salt irrigations vs saline irrigations with nasal steroids for symptomatic treatment of chronic rhinosinusitis: A randomized, prospective double-blind study.» Int. Forum. Allergy Rhinol., vol. 2, pp. 252-257, 2012.
- Koksali T. et al., «Comparison between the use of saline and seawater for nasal obstruction in children under 2 years of age with acute upper respiratory infection.» Turk. J. Med. Sci., vol. 46, pp. 1004-1013, 2016.
- Singh R. et al., «Comparative study of intranasal hypertonic.» Int. J. Otorhinolaryngol. Head Neck Surg., vol. 3, no. 1, pp. 104-107, 2017.
- Strnad P. et al., «Efficacy of nasal irrigation with 2.2% hypertonic seawater in the treatment of chronic rhinosinusitis in adults.» Rev. Laryngol. Otol. Rhinol., vol. 137, no. 1, pp. 3-10, 2016.
- Rapijko P. et al., «The use of a hypertonic sea water solution in patients after surgical treatment of the nose and paranasal sinuses.» Otolaryngol. Pol., vol. 64, no. 1, pp. 29-30, 2010.

Année / Anno / Jahr / Jahr : 2007.

LOGO	FR	IT	NL	DE
	Fabricant	Produttore	Fabrikant	Hersteller
	Distributeur	Distributore	Verdeler	Vertriebshändler
MD	Dispositif médical	Dispositivo medicale	Medisch hulpmiddel	Medizinisches Gerät
CE	Logo de conformité CE avec numéro d'identification de l'organisme notifié	Logo di conformità CE con numero di identificazione dell'organismo notificato	CE-conformiteitslogo met identificatienummer van de aangemelde instantie	CE-Konformitätssymbol mit Identifikationsnummer der benannten Stelle
	Lire attentivement les instructions d'utilisation	Consultare le istruzioni per l'uso	De gebruiksinstructies raadplegen	Beachten Sie die Gebrauchsanweisung
	Date limite d'utilisation	Data di scadenza	Houdbaarheidsdatum	Mindesthaltbarkeitsdatum
LOT	Numéro de lot	Numero di lotto	Lotnummer	Chargennummer
	Dès 0 mois	Dai 0 mesi	Vanaf 0 maanden	Ab 0 Monaten
3	Conformité à la Directive relative aux générateurs d'aérosols	Conformità con la direttiva sui generatori di aerosol	Naleving van de richtlijn inzake aerosolgeneratoren	Einhaltung der Richtlinie über Aerosolgeneratoren
175	Contenance du récipient	Capacità del contenitore	Inhoud van flacon	Inhalt des Behälters