DE

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EN

CE 1370  
**Hemorrhoid Gel**  
 Medical Devices Class IIa

BEHREND HOME CARE® Hemorrhoid Gel is based on **Hyaluronic acid**, which is present in high amount in the extracellular matrix, and is therefore able to create an optimal microenvironment that help tissue repair.

**Its light, fresh and delicate gel formulation, based on natural ingredients**, offers immediate comfort by performing an excellent lubricating action.

The treatment is enriched with Trehalose that helps to restore the correct skin hydration levels and the physiological elasticity of the anal mucosa.

BEHREND HOME CARE® Hemorrhoid Gel reduces the rubbing with feces, protecting from irritation and perianal congestion, and allowing a rapid relief from discomfort.

BEHREND HOME CARE® Hemorrhoid Gel reduces and alleviates pain, itching and burning sensations.

#### INDICATIONS

Indicated in **anorectal disorders processes** such as: **internal and external hemorrhoids, anal fissures, proctitis and anal itching.**

Ideal as a prolonged treatment of chronic symptoms and in cases of:

- prolapse of the tissues
- constipation
- tendency to recurrence

Suitable to prevent anus-rectal discomfort, it maintains physiological tone and elasticity.

**Cortison free formulation.**

#### CLINICALLY TESTED FORMULA

BEHREND HOME CARE® Hemorrhoid Gel has been dermatologically tested

#### DOSAGE AND INSTRUCTIONS

For perianal and/or endorectal application. Wash carefully the affected areas before application. Apply two or three times a day after each evacuation. Continue the treatment for 1 – 2 weeks until symptoms disappear. After application it is advisable to avoid evacuation for at least 20-30 minutes to avoid early removal of the product.

#### HOW TO USE

Unscrew the tube cap, pierce the safety seal with the back of the cap and screw the anus-rectal cannula neck of the tube itself. Remove the cannula cap and squeeze the tube until the cannula is filled with gel. Pay attention to release a small amount of product to lubricate the cannula facilitating the application. Insert the cannula into the rectum.

Squeeze the tube again until a sufficient quantity of product is introduced. Apply also on external area. After use, wash carefully the cannula, let it dry and cover it with the cap. Make sure that the tube is sealed appropriately.

#### COMPOSITION

Hyaluronic Acid, Trehalose, Methylsulfonfylmethane, Glycerin, Hydroxyethylcellulose, Bisabolol, Water, Sorbitol, Pentylene Glycol, Propylene Glycol, Disodium EDTA, Sodium/Calcium Copolymer, Triclosan, Symdiol (1,2-Hexanediol, Tropolone, Caprylyl Glycol), Sodium Hydroxide.

#### ASPECT

Gel available in 30 ml tube with multi-use cannula for internal application.

#### WARNINGS

The device is not subject to special precautions of use, we advise however to:

- Keep out of the reach of children;
- Do not exceed the recommended dose;
- Do not use after the expiry date; the expiry date refers to the intact and properly stored product;
- In case of adverse reactions, suspend the use of the product and consult a doctor in order to fix a proper therapy;
- No data to be reported on interactions with drugs normally used in the treatment of hemorrhoids or anal fissures.
- Do not use the product for more than 30 consecutive days

Consult your doctor in case of use during pregnancy or breastfeeding.

Store at room temperature away from heat sources. Compliance with the instructions contained in the leaflet reduces the risk of side effects. It is important to inform your doctor or pharmacist about any side effect not listed in this leaflet.

#### CONTRAINDICATIONS

Do not use in case of hypersensitivity reactions to the ingredients.

Rev 00 June 2022

PZN 18204163

The Medical Device must not be thrown into waste water or household waste.  
 Ask the pharmacist how to dispose of the Medical Devices that are no longer used.  
 This will help protect the environment.

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