OKKLUglas®**Zweckbestimmung:**

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OKKLUglas®**Intended use:**

OKKLUglas® is a ready-to-use eye dressing (watch glass dressing) with all-round adhesive zone and transparent, break-proof protective cover for covering the eye after surgical procedures, injuries or impaired eyelid closure.

Application:

To ensure optimum adhesion, the skin should be clean, free of grease and dry. Bushy eyebrows can impair the adhesion of the patch: Shave or pluck the eyebrow if necessary. Also available in the EXTRA version: super hold thanks to extra large comfort patch.

Carefully remove the cover film. Place the watch glass bandage in the middle of the eye so that the unbreakable protective cover is centered over your eye and your eye is not covered by the plaster. For the EXTRA version: The large adhesive surface points upwards towards the eyebrow. Apply the plaster to the skin with gentle pressure.

Please consult your doctor regarding medication, the optimum variant and the appropriate size.

Specifications for proper use:

Variation	Description	Size	Performance features	Indications
OKKLUglas®	with comfort plaster	93 x 74 x 18 mm	mechanical barrier function, Formation of a moisture chamber	after ptosis surgery, for lagophthalmos, facial nerve palsy, to treat injuries to the outer eye, for protection after eye surgery
OKKLUglas® SILICONE	Plaster with silicone adhesive	93 x 74 x 18 mm		
OKKLUglas® baby SILICONE		48 x 39 x 11 mm		
OKKLUglas® kids SILICONE		70 x 58 x 14 mm		
OKKLUglas® EXTRA	with extra large comfort patch	93 x 90 x 18 mm	mechanical barrier function, optimum air circulation through ventilation openings	after ptosis surgery, lagophthalmos and after eye surgery as mechanical protection for the eyes
OKKLUglas® AERO	with comfort plaster	93 x 74 x 18 mm		
OKKLUglas® AERO SILICONE	Plaster with silicone adhesive	93 x 74 x 18 mm		
OKKLUglas® kids AERO SILICONE		70 x 58 x 14 mm		
OKKLUglas® baby AERO SILICONE		48 x 39 x 11 mm		
OKKLUglas® AERO EXTRA	with extra large comfort patch	93 x 90 x 18 mm	Mechanical barrier function, optimum air circulation through ventilation openings, UV protection class 2-1.2 according to DIN EN 170 due to UV-stabilized plastic component	after ptosis surgery, lagophthalmos and after eye surgery as mechanical and UV protection for the eyes
OKKLUglas® AERO UV	with comfort plaster	93 x 74 x 18 mm		
OKKLUglas® AERO EXTRA UV	with extra large comfort patch	93 x 90 x 18 mm		

Contraindications:

Allergy to patches / patch intolerance

Warnings / restrictions:

Wearing a watch glass bandage can severely impair vision (ability to drive). The plaster must not be applied to injured skin or mucous membranes.

 Due to the risk of increased bacterial contamination and loss of complete adhesion, this product is intended for single use only.

Patient target group:

Adults, babies and children (with adult assistance during application)

Vigilance:

Serious incidents that have occurred in connection with the product must be reported to us as the manufacturer and/or the competent authority of your Member State.

Storage instructions:

 Store in a dry place  Protect from sunlight

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