# EULATIN® NH Salbe

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Stand der Information: 03/2017

# EULATIN® NH Ointment

Medical device to relieve bowel movement when suffering from haemorrhoids

## INSTRUCTIONS FOR USE

### 1. Therapeutic purpose

*Eulatin® NH Ointment* achieves a physical relief of bowel movement by lubricating the rectum, the anus, and protects the imperforate anal mucosa in case of haemorrhoids. The product improves the sliding properties of stool and protects the mucosa from laceration and itching.

### 2. Therapeutic indications

*Eulatin® NH Ointment* is ready to use for internal and external haemorrhoids in the end region of the rectum and of the anus for the physical protection of the imperforate anal mucosa in adults and for the prevention of pathological itching.

The product is designed and suitable for the professional care of patients, the handicapped and the elderly, and for domestic use.

### 3. Undesirable effects

Undesirable effects are not known for this product when applied as prescribed.

### 4. Histocompatibility, Interactions

No data is available that suggests there may be a histoincompatibility of the product.

*Eulatin® NH Ointment* should neither be applied in combination with other creams and ointments in the anal areas nor with drugs applied rectally.

### 5. Contraindications

*Eulatin® NH Ointment* may not be applied when an allergy to one of the ingredients are known.

#### Attention:

The product may not be applied during pregnancy or lactation.

### 6. General special precautions

The product should not be applied if blood is discharged or blood is suspected in stool and if there is simultaneous occurrence of pain and fever.

The product is solely designed for application to the treatment area and may not enter the body by ingestion.

#### Attention:

Keep out of the reach and sight of children.

#### Attention:

When using *Eulatin® NH Ointment* in the area of the anus and at the same time using condoms of latex, the tensile strength may be reduced and thus lead to reduced protection from the condom due to the excipients petrolatum and paraffin.

### 7. Product properties

Haemorrhoids cause discomfort which, depending on the severity of the disorder, may lead to itching, bleeding, inflammation or thrombosis. Severe pain is experienced particularly in bowel movement. As a reflex to the severe pain experienced in bowel movement, unconditioned constipation occurs, and more intensive pressing strain is required for bowel movement. This then leads to a further amplification of existing haemorrhoids resulting in even more pressing strain being required.

*Eulatin® NH Ointment* lubricates the anal passage and gives protection against haemorrhoids. The stool can slide more easily through the pre-lubricated anal

passage, bowel movement therefore requires little pressure and painful pressing can be avoided. In addition, the anal mucosa is optimally protected against injury by the covering properties of the product.

### 8. Product application

The anal areas as well as your hands should be cleaned carefully before each application of *Eulatin® NH Ointment* with tepid water.

In the case of external haemorrhoids, apply directly 2-3 cm of ointment with cleaned and unijnured fingers.

For application to the rectum (for internal haemorrhoids) screw the enclosed applicator (accessory) to the tube and lubricate the tip with the product. Then insert the applicator carefully into the anus and apply the product in a rotary movement inside the anus by pressing the tube once/twice. Clean the outside of the applicator after use carefully with tepid soap water. It is advisable to cover the application area with cellulose or something similar to protect clothing.

Apply *Eulatin® NH Ointment* by inserting applicator into the anus two to three times a day or before bowel movement, respectively.

The continued application should not exceed four weeks.

Used containers should be disposed of separately in domestic waste, expired containers or residuals should be brought to chemist's for disposal.

### 9. Physical product data

Composition:

*Eulatin® NH Ointment* contains:

White petrolatum, lanolin, lanolin alcohol ointment, medicinal yeast, purified water, medium-chain triglycerides, hard paraffin, highly-disperse silicon dioxide, tannic acid, alkaline bismuth gallate, fluid extract of hamamelis bark.

Appearance, odour: yellowish ointment, odourless

Shelf-life: 2 years

The shelf-life is printed on the tube and the outer package of the product.

Special precautions for storage:

Store *Eulatin® NH Ointment* at room temperature but not above 25 °C. Close the tube tightly after usage.

### 10. Delivery specifications

The outer packaging of the product contains:

Tube with 30 g or 60 g contents

Accessory: Applicator for internal application in the rectum.

### 11. Manufacturer

Manufacturer of the medical device:

Euro Vital Pharma GmbH  
Borsteler Chaussee 49  
22453 Hamburg  
Phone +49 40 529526-900  
Email info@eurovitalpharma.com

#### Note:

Please read instruction for use

Manufacturer

Batch

-25 °C

Temperature limitation

EXP

Please read instruction for use

This leaflet was last approved in 03/2017.

1142034/20

CE 0482